

2201 / 2205 series / 2206 series / 2207 / 2242 Beauty Beds

Welcome to use our Beauty Beds.A1
Please read the user manual carefully before using the beds.

1、About This Manual


This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, SILVERFOX will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of our company SILVERFOX.

2、Warning

- Do not stand on this equipment.
- Children are not allowed to play with the equipment. Please adjust the backrest to the highest position when adults are not around.
- Do not place the product in humid environment, to avoid malfunction.
- Equipments should not be operated under painting/coating or supplying oxygen.
- Unless being monitored or instructed by personnel who can be responsible for their personal safety, the people who have physical obstacles, tactile disorders, mental disorders, lack of experience, common sense or children, are not allowed to use the equipment.
- Please make sure there is no accessible thing within the stroke during any operation of the beds.
- Operation should be processed by trained professionals.

3、Precautions

- The equipment should be in charge by an experienced person who must be familiar with the product structure, performance and use methods. And make the regular maintenance work.
- Read this manual carefully before installation, operation and adjustment. And please keep it well.
- This product shall not be used for single side or single point loading work.
- Comply with safety operation rules, do not overload the product, ensure that the product safety protection equipment is complete and reliable, to eliminates unsafe factors on time.
- Tighten bolts and nuts regularly to avoid failure caused by loose, sliding, vibration and falling parts.
- During the maintenance, the operator shall not disassemble any parts of the bed at will.
- This product is not allowed to operate when it's with fault. If abnormal phenomenon or abnormal sound is found, it must be stopped for inspection immediately, and work can be resumed after troubleshooting.
- The operator must be familiar with the precautions described in each section of the manual to ensure the safety of person and product.
- Please operate the equipment strictly according to the warning label pasted to the products.
- Maintenance should be processed by trained professionals.
- After each treatment, please use neutral reagent to clear the dust, stains on leather or on wood pieces.
- When adjusting the backrest, seat cushion, footrest or lifting of the whole bed by hand controller or installed foot controller, please keep your hands, feet and head away from the position as the below arrow shown. (Please see right picture)

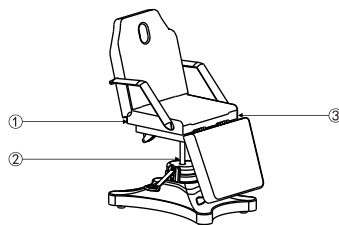

4、Applicable Range

These beauty beds are suitable for beauty, massage and manicure treatment.

5、Installation


1. Put the bed to a proper location according to the overall layout, the daylighting or artificial lightings and operation condition of the salon's or the home. And keep it in clean, dry and cool environment.
2. Open the packing carton to check if the bed is intact and whether the accessories or spares are complete. Please contact distributor or manufacturer if there's any problem.
3. Installation Guide for Beauty Bed
Model 2205/2205B/2206/2206A/2207/2242

Place the bed to the proper position. Please note that it is unnecessary to install extra screw on the floor to fasten the bed, because stability performance has been taken into consideration.


Please ensure the bed is placed on a firm and flat ground, so as to any avoid accident.

If the bed rotates or slant due to rough ground, please adjust the four pieces of plastic feet (M10×30) and nuts (M10) by hexagon open-end wrench, which are to fasten on the bottom panel, so as to ensure the bed is stable.


2201

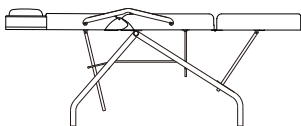


2205

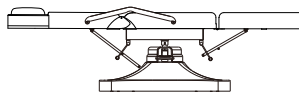


2205B

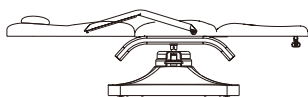


2206/2206A

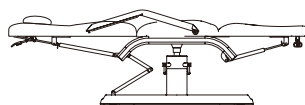


2207

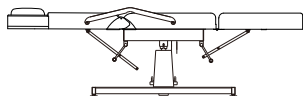


2242

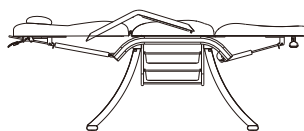

2201 Installation procedure

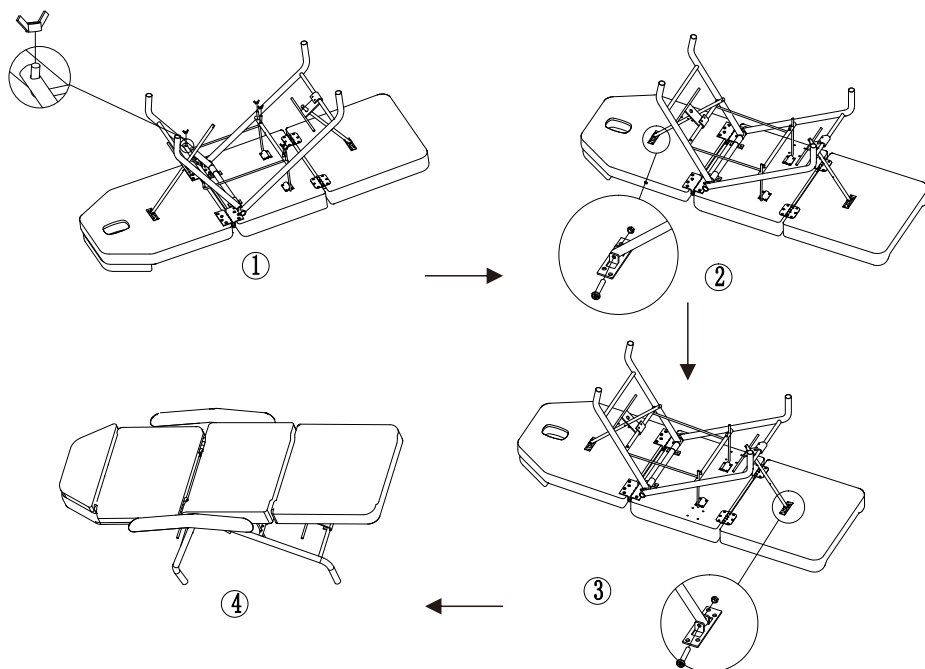


2205B / 2206 / 2206A Installation procedure

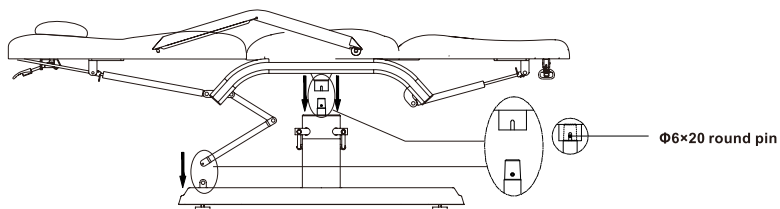


2205 Installation procedure


Pic 1

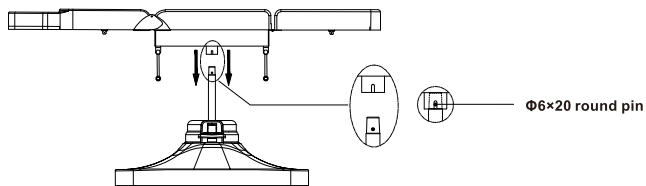


Pic 2

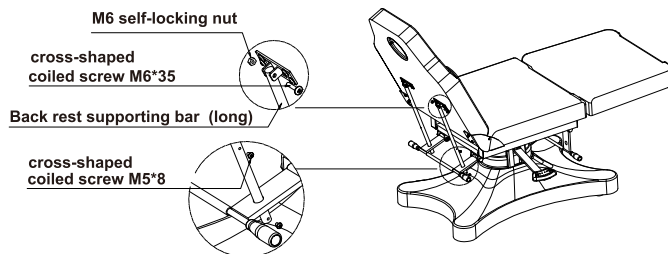

Pic 3

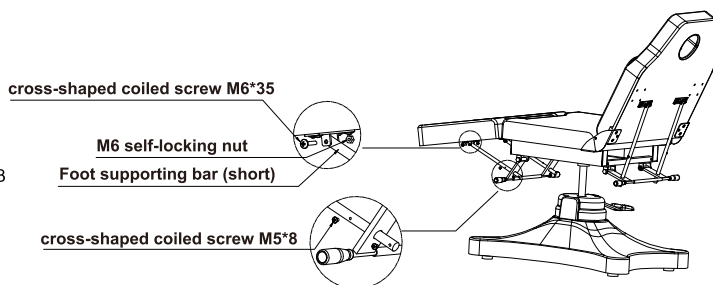


Pic 4

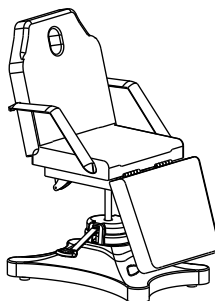


2207

Installation procedure


Pic 1

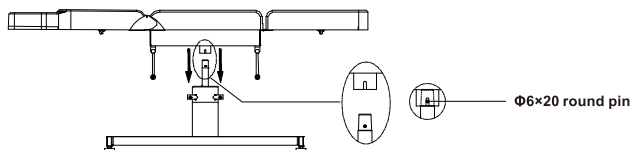


Pic 2

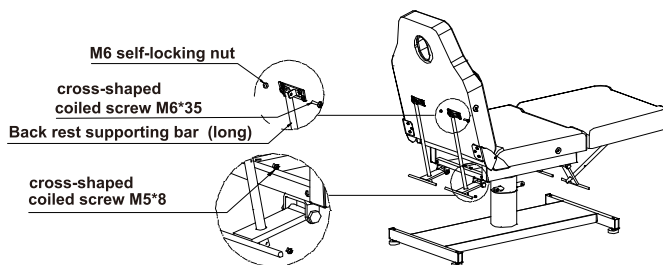

Pic 3

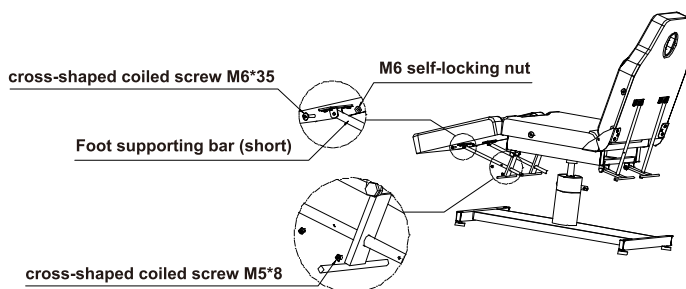

Pic 4

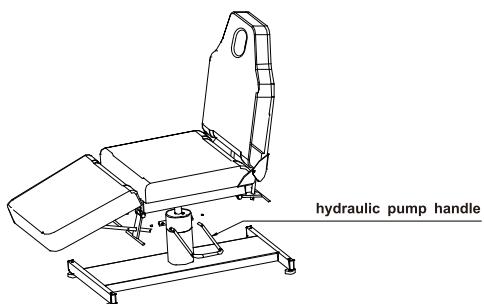


2242 Installation procedure

Pic 1



Step 1, Raise up the TOP part of bed and overturn it, see pic1&2.

Pic 2

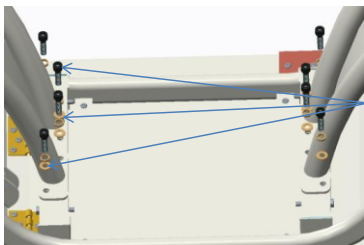


Screw the washer, spring washer and hexagon socket screw aimed to the screw hole.

Step 2, Screw the washer, spring washer and hexagon socket screw and aim to the screw hole, see pic 2.

Pic 3


Install the two metal bars with M6X30 hexagon socket screws and plastic washers.

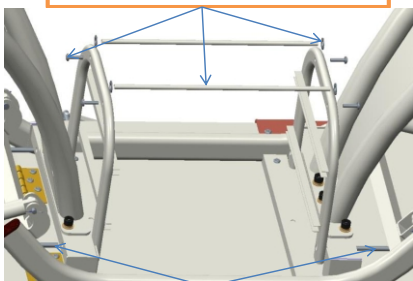

Screw the lateral supporting metal poles with M8X60 hexagon bolt.

Step 3, Screw the lateral supporting metal poles with M8X60 hexagon bolt, then install the two metal bars with M6X30 hexagon socket screws and plastic washers, see pic 3.

Pic 4

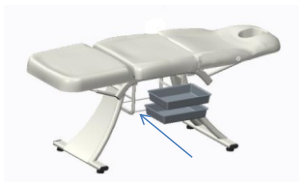

Step 4, Install the shelf, see pic 4.


Pic 5




Step 5, Install the armrest and headrest, see pic 5.

Install the armrest as below.

2201 / 2205 / 2207

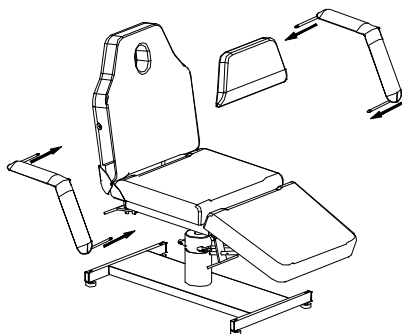


2205B / 2206 / 2206A / 2242

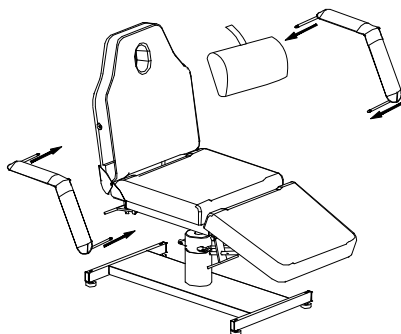

2205 / 2205B / 2206 / 2206A / 2207

Height adjusting

1. Step slightly on the foot pedal to raise the whole bed. Step hard on the foot pedal constantly to lower the whole bed.
2. Pull the pedal to the highest position to make the bed fixed.

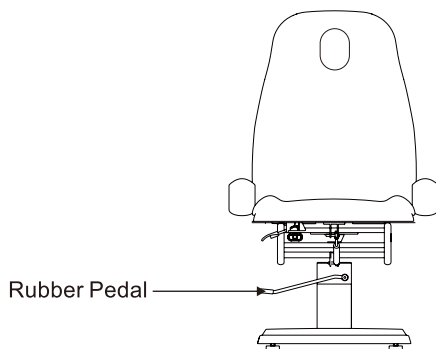




This product graphic is only for display, the specific product structure please refer to the actual product.

6、Technical Parameters

Environment Temperature	0°C~+40°C
Relative Humidity	30%~90%
Lifting Rated Load	175 kg




2201

Package Size

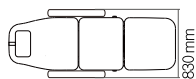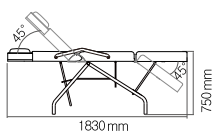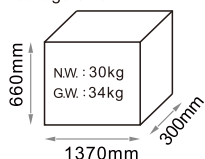




2205/2205B

Package Size

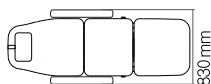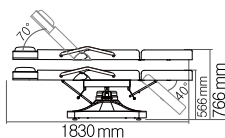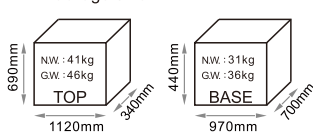




2206/2206A

Package Size

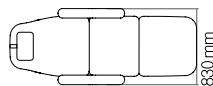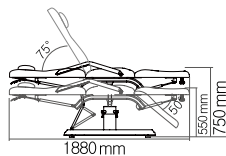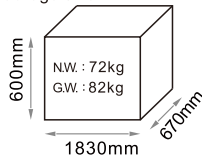




2207

Package Size

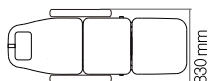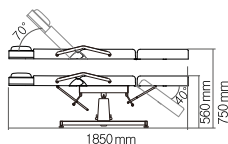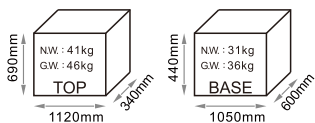

2242

Package Size

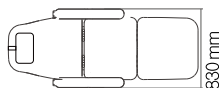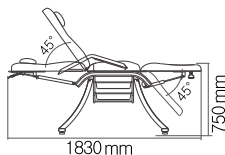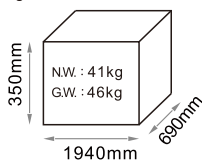

7、Maintenance

- 1.After every treatment, please only use neutral solution to clear the the dust, stains on leather, metal part or plastic cover. Disinfecting solution like alcohol, acid or alkaline solution are forbidden, which is to prevent leather or plastic cover from aging too fast.
- 2.Please lubricate the moving parts regularly (once a year)

8、Troubleshooting

Abnormal Sound	1. The rotating parts are short of lubricating oil.	1. Add lubricating oil.
	2. Rotating positions (such as hinge, POM sets) are short of lubricating oil.	2. Add lubricating oil.

If problem is still not solved after above operations,
please contact maintenance center or supplier to solve it.



Do not dispose of old or defective appliances in domestic garbage;
this should only be done through public collection points.

▲ Please check the product specification before use. This manual is for
reference only. Specifications are subject to change without prior notice.